



Development Company LLC™

conformaMed®

The conformaMed cushion system uses specially designed shape memory elastomer foams selected for their naturally conforming Body Sensitive™ properties, durability, and light weight. Properly installed, your conformaMed cushion will provide both pressure management and postural stability when used with various types of mobility equipment and standard seating. The all-way stretch cover is moisture and fluid proof and should be left on at all times except for cleaning. The bottom of the cushion features a rugged, non-skid surface. An added option is the velcro lock-pad which helps keep the cushion in place.

Care and Use Guide

Covers

To remove the cover (s), use the zipper at the rear of the cushion. Bend the cushion slightly along its width and slide the foam insert out. To reinstall, reverse these steps, being careful to insert the foam cushion with the dark gray foam on the bottom of the cover. The cover may be cleaned in using a standard automatic washer, using warm water, delicate cycle and a suitable detergent. Do not dry the cover in an automatic clothes dryer. Allow the cover to air-dry before reinstallation.

Avoid using pins to attach items to the cover, as this will puncture the fluid proof liner and may allow fluids to enter.

Cushion

The special foam materials in your conformaMed cushion are for use primarily indoors. The cushion may be used outdoors in cold weather, however to ensure best performance, keep at room temperature until ready for use. If the cushion is exposed to temperatures below 32° F (0 C) for long periods, allow it to come to room temperature (65°F/18C and above) before using. In very cold temperatures, the cushion may temporarily feel stiffer than normal.

The cushion is not washable- **do not** put it into a washing machine.

Placement

Place the cushion with the carry-handle toward the front, and the zipper at the rear. Depending on body weight, your conformaMed cushion may require a few minutes to adjust its shape for optimal comfort.

To use the Velcro lock-pads supplied on selected cushion bottoms, remove the paper release liner to expose the pressure sensitive adhesive, and carefully place the cushion on the chair in the desired position. Apply pressure to the cushion to ensure the lock-pads are securely fastened before using.

If you have any questions about your conformaMed cushion, please contact your supplier or call us at 269.345.1620.

Limited Warranty

Helvetia Development Company, LLC warrants to the Original Purchaser of this product that the product is free from defects in material and workmanship under normal use and service for 18 months. If this product is found to be defective in material or workmanship under normal use and service during this period, Helvetia Development Company LLC will, at its sole option, repair or replace the product without charge, provided that the original purchaser returns the product, with the original receipt, shipping prepaid, to: Helvetia Development Company, LLC 225 Parsons St., Kalamazoo, MI 49007, along with a letter indicating the specific reason for returning this product.

This warranty does not apply to, and is void as to defects or physical damage resulting from abuse, neglect, improper repair, alterations, modifications, or use contrary to that intended by the manufacturer, except to the extent prohibited by applicable law. Any implied warranty of merchantability or fitness for a particular use or purpose is limited in duration of this limited warranty. Helvetia Development Company, LLC shall not be liable for any incidental or consequential damage arising from any breach of any express or implied warranty on this product.

Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion might not apply to you. This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.



225 Parsons St. Kalamazoo, MI 49007
Phone 877.626.5704 Fax 269.345.1748